

## 1. General information

### 1.1 Description

- a. Repetition rate adjustment wheel
- b. Repetition rate indicator
- c. Needling trigger and indicator
- Red LED : Standby
- Green LED : Flashes green when needling
- d. Needle length adaptor
- e. Needle cartridge
- f. DermaJet power socket
- g. DermaJet adaptor

### 1.2 Box contents

Please verify that the box contains the following:


- a. Handpiece
- b. Adaptor
- c. 1 x sterile needle cartridge
- d. 5 x needle length adaptor
- e. 1 x dust cap
- f. User manual / warranty conditions

## 2. Assembly

### 2.1. Preparation



Remove the dust cap by rotating it anti-clockwise and pulling it off.

It is very important that you keep the dust cap on the device when not in use.

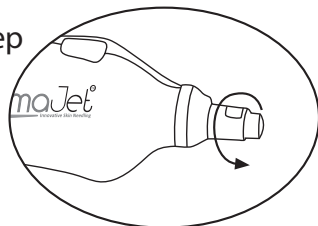

### 2.2 Needle length selection

The needle length is selected by using different needle length adaptors for each needle size, ranging from 0.2 to 2 mm. Depending on the required needle length, the suitable adaptor is inserted and rotated clockwise to lock.

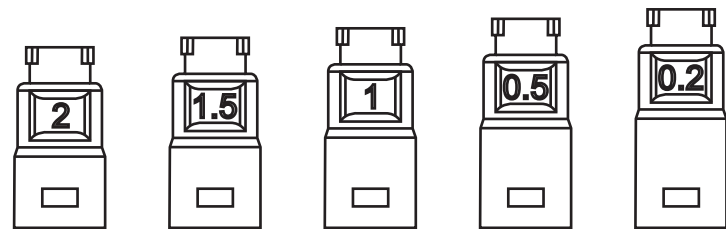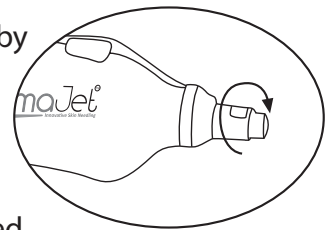

### 2.3 Mounting the needle cartridge

Take the needle cartridge out of its sterile packaging. Put the needle cartridge on the end of the needle length adaptor and rotate clockwise to lock.

## 3. Operation

Insert the coiled cable socket into the DermaJet power adaptor and the power cable into a mains socket. Select the needling frequency. The frequency is selectable from 3 to 15 punches per second (Hz). Consult your vendor's usage guidelines for the optimal frequency, needle length and number of needles. The device will work as long as the needling trigger is pressed, or it will work for 1 sec. if the needle trigger is pressed just once. When "0 Hz" is selected, pressing the trigger has no effect.

## 5. Do not use in case of

- Immune depression, acute illness
- Allergy to stainless steel
- Hemophilia
- Melanosis
- Using blood-thinning drugs
- Children

## 6. Warnings

**a.** Please only use with the original mains supply for your country.

**b.** Store the device between 10-35 °C / 50-95 °F.

**c.** Do not attempt to repair the device yourself.

**d.** Do not touch the device with wet hands and do not use the device if you see any cuts or damage to the cable or the unit. Do not leave unattended while plugged into the mains supply.

**e.** It is the user's responsibility to use the device according to this guide and all available instructions, especially the vendor's needle-cleaning and disinfection guidelines as well as dermaneedling instructions.

**f.** The manufacturer and vendor are not liable for the results obtained by the device.

## 7. Cleaning and maintenance

WHEN THE DEVICE IS NOT IN USE, PLEASE MAKE SURE THAT THE DUST CAP IS IN PLACE. DUST MAY INTERFERE WITH THE NEEDLE LENGTH ADJUSTMENT, WHICH MAY CAUSE UNINTENDED EFFECTS.

Diluted alcohol and a soft cloth can be used to clean the device thoroughly.

Please do not use other chemicals or acids for cleaning.

Certification  
CE class 2a device

## 8. Warranty and Replacement

Each DermaJet is under warranty for 12 months after purchase. Faulty devices will be replaced in this period.

## 9. Warranty conditions

Wrong use, liquid spills, damage due to physical impact, normal wear & tear and any other problem not caused by obvious manufacturing flaws are not covered by the warranty.

