


Teeth whitening pen 35% Carbamide Peroxide

Dentist-grade extra strong formula teeth whitening pen. This pen really works! Safe and easy for home use. [No harmful ingredients](#) such as Chlorine dioxide. The gel is transparent so you can use it at work, in school etc.

1. Brush your teeth and wait at least half an hour.
2. Uncap the pen and turn the end part until gel appears at the tip.
3. Then apply a thin layer to your teeth.
4. You can use a finger nail to shield your gums as you apply the gel.
5. Let it dry with open mouth for at least half a minute.
6. Do not let your lips touch the gel as it dries.
7. Wait at least half an hour before you eat anything.
8. The dried gel will be removed with the next toothbrushing.

35% Carbamide Peroxide. Comparable to 12% Hydrogen peroxide but much safer to use. For best results, treat twice a day for a couple of weeks.

- Avoid contact with the gums by applying in front of a mirror and "painting" in the direction away from the gums. The advantage with a tooth whitening pen is that you can target teeth that need extra whitening, dark spots etc. and you can avoid crowns and teeth that do not need whitening. There is no risk of gum sensitivity when you do not apply it to the gums.
- There is no danger of "overbleaching" when you for example use it overnight, because after an hour or two, the gel simply stops working. You need quite a few applications to get several shades lighter, so the process is gradual and you can adjust the application until your teeth are evenly lighter.
-  The only thing you have to really be careful with is not to brush the gel onto your gums and if you do, rub it off immediately with a wet tissue. Take this advice seriously because if you repeatedly get this on your gums and you don't remove it, the gel can damage the gums. The gel won't "drip" though, so as long as you pay attention during the application, all will be well. This pen is "dentist-strength" and that means great results but with that whitening power comes the need for careful application.

Contrary to Hydrogen Peroxide whitening pens, Carbamide Peroxide teeth whitening pens can be stored at room temperature and can be used for at least a year after purchase.